

Encore® HydraSoft™

New!

The Encore® HydraSoft™ powder-free latex surgical glove is designed to retain moisture and rehydrate your skin, despite continual glove-wearing and frequent contact with astringent hand cleaners and disinfectants. Encore® HydraSoft™ helps keep your skin moist and supple by fulfilling its natural barrier function.

Safety & Protection

Infection Control

Viral Penetration. Although not an FDA requirement, Ansell's premium line of surgical gloves is tested to ensure protection against viral permeation. Encore® has been tested and passed the viral penetration according to ASTM F1671-97a.

Cuff Roll-Down. This glove has Ansell's patent-pending SureFit technology, which provides surgeons, nurses and other healthcare professionals with superior barrier protection during operating room procedures.

Bone Cement. As long as the preparation and use directions on the bone cement are followed, this glove provides a protective barrier for handling mixed bone-cement product.

Skin Protection


Low Protein Content

Lowry Protein Assay	Typical Results (µg/g)
Avg.	29.5
Min.	22
Max.	37

This glove contains 50 µg/g or less of water/extractable protein per gram.

Caution: Safe use of these gloves by latex-sensitized individuals has not been established.

Moisturizing. Encore® HydraSoft™ utilizes two moisturizers: skin-friendly glycerine, which attracts and retains water, and dimethicone, which prevents the skin from drying out.

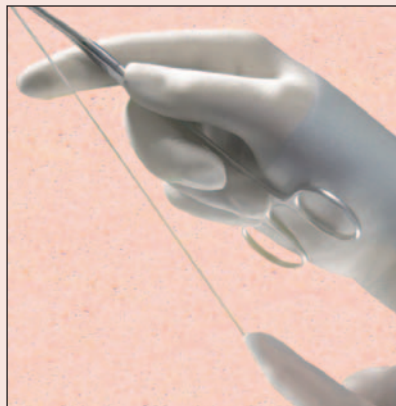

Comfort & Fit

Donning

Alcohol Resistant. Ansell's proprietary polyurethane inner coating will not break down. It is resistant to alcohol-based hand disinfectants and adds another layer of safety between your skin and the latex.

Damp-Hand Donnability. The HydraSoft™ coating increases the ease of damp-hand donnability.

Double Donnability. This glove is recommended to be used as an under glove because of its HydraSoft™ Technology incorporated in this product.

Hand Fatigue

Shape of Glove/Former. Designed with Anatomic Curved Former for a more natural fit and less fatigue. This exclusive Ansell former is the result of a staff dedicated to the development and continuous improvement of formers.

Stretchability (Modulus). Because this glove is latex, wearers experience a more secure fit that provides greater comfort without bagginess.

Tactile Sensitivity

Dependability. The thickness of these gloves is optimized for the best balance of protection and tactile sensitivity, all while allowing for a comfortable fit during the majority of surgical procedures.

Grip

Application Driven. Anti-slip smooth finish prevents slip.

HYDRASOFT™

Ansell

Everything you touch...we touch.